

## The SaltFacial Gliding Gel

### Active Ingredients:

- Plant stem cells & Citrus Limonium SWT-7 + Naolys Fiber Booster Citrus

Intended Use - hydrating and soothing gel; coupling agent for ultrasound and RF treatments.

### INSTRUCTIONS FOR USE:

1. Apply approximately ½ - 1 tsp (3-5 pumps) to the treatment area – avoiding the eyes and any areas of sensitivity.
2. Gently ‘press’ into treatment area until absorbed.
3. Apply approximately 1–2 Tbs of The SaltFacial™ Gliding Gel to the treatment area. Additional Gliding Gel may be added as needed during treatment.
4. Once treatment is complete, remove excess gel with a soft, damp cloth.

